



SeeClear[®]

Laparoscopic Smoke Evacuation Systems

Features and Benefits

- Maintains clear field of vision
- No involvement after easy setup
- Optimal flow rate is pre-set, maintaining pneumoperitoneum
- Traps smoke and aerosolized pathogens
- Absorbs odors and chemical toxins



The Easiest Solution for a Smoke-Free Laparoscopic Procedure

The SeeClear® Max™ (green filter housing) and SeeClear® XCL™ (pink filter housing) are passive disposable multi-stage filter systems. The ULPA filter traps smoke, particulates and aerosolized pathogens. An activated charcoal membrane absorbs odors and chemical toxins.

The elevated intraperitoneal pressure pushes out the smoke, etc. along with the CO₂. No vacuum pump is required – it functions silently. No extra tubing – one easy connection.


Flow rates are pre-set to optimize smoke removal without losing pneumoperitoneum.



Three Easy Steps for Use



Attach the SeeClear to the side port of the trocar that is **largest and closest to the source of smoke or mist.**

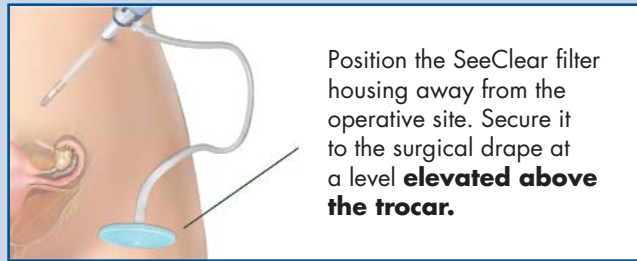

Position the SeeClear filter housing away from the operative site. Secure it to the surgical drape at a level **elevated above the trocar.**

NOTE: Elevating the SeeClear filter prevents condensation from collecting in the filter housing and reducing the flow rate.



Open the side port of the trocar to its full open position **before smoke or mist is created.** The SeeClear filter is not a catch-up device.

Product Specifications

Product Number

SC062500

Description

SeeClear® MAX™ (green filter housing)
Maximum flow rate 6.0 liters/min at 15 mmHG pressure differential

SC082500

SeeClear® XCL™ (pink filter housing)
Maximum Flow Rate 8.0 Liters/min at 15 mmHG pressure differential

Both Models

Connector:	Acrylic Standard Luer
Tubing:	30" length, PVC
Filter Casing:	Medical Grade Polypropylene
Filter Material:	Deodorization – activated charcoal Filtration – ULPA Grade Hydrophobic Glass Microfiber Efficiency – 99.999% of 0.1 micron particles (aerosolized)
Drape Clamp:	ABS, Porene GA850
Double Loop Tether:	PVC
Latex-Free	
Single Use Only	

CooperSurgical

coopersurgical.com • also on YouTube at youtube.com/coopersurgical

SeeClear®

Laparoscopic Smoke Evacuation Systems

95 Corporate Drive, Trumbull, CT 06611
800.243.2974 • 203.601.5200
Fax: 800.262.0105

SeeClear® is a registered trademark of CooperSurgical.

*U.S. Patents: 6,110,259; 6,589,316 B1; & 6,881,236 B2; 7,258,712; EP Patent 1039961 & other Int'l patents; patent pending

Form #81697 • Rev. 02/10
© Copyright 2010 CooperSurgical, Inc.